

Regenerative Application Guide

PROCEDURAL SOLUTIONS FOR YOUR REGENERATIVE NEEDS

		SOFT TISSUE AUGMENTATION	BLOCK GRAFTING	RIDGE AUGMENTATION "SANDWICH" TECHNIQUE ^{1,2}	SINUS LIFT (PER SINUS)	3-WALL EXTRACTION SOCKET	4-WALL EXTRACTION SOCKET	IMPLANT PACKING / PERIODONTAL DEFECT
<i>Puros</i> ® Cancellous Particulate Allograft			✓ (0.5cc) (To fill around edges)	✓ (1 cc) (Inner)	✓ (2cc)	✓ (0.5-1cc)	✓ (0.5-1cc)	✓ (0.5-1cc)
<i>Puros</i> Cortical Particulate Allograft				✓ (1 cc) (Inner)	✓ (2cc)			
<i>Puros</i> Demineralized Bone Matrix Putty			✓					✓
<i>Puros</i> Demineralized Bone Matrix Putty with Chips			✓		✓	✓	✓	
<i>Puros</i> Block Allograft			✓ (10-15mm)					
CollaPlug® Absorbable Wound Dressing	 Solid						✓	
CollaCote® Absorbable Wound Dressing					✓ (For Schneiderian membrane tears)			
CollaTape® Absorbable Wound Dressing		✓			✓			
<i>Zimmer</i> Socket Repair Membrane						✓		
<i>Zimmer</i> Collagen Capsules	 Hollow				✓	✓		
<i>Zimmer</i> Collagen Wedge					✓			
<i>BioMend</i> ® and <i>BioMend</i> ® Extend Absorbable Collagen Membranes (RIGID)				✓	✓ (30x40mm)			✓
<i>Puros</i> and <i>CopiOs</i> ® Pericardium Membranes (PLIABLE)			✓ (30x40mm)	✓				
<i>Puros</i> Dermis Allograft Tissue Matrix		✓						
<i>Zimmer</i> CurV™ Pre-Shaped Collagen Membranes				✓				



Zimmer Dental Soft Tissue Options

FORMULATION	WOUND DRESSINGS		BARRIER MEMBRANES		SOFT TISSUE GRAFTS	
CLINICAL NEED	Wound healing	Wound stabilization	Space creation and maintenance		Adaptability	Soft tissue augmentation
ZIMMER OPTIONS	CollaPlug® CollaCote® CollaTape®	HemCon Dental Dressings	<i>BioMend</i> and <i>BioMend Extend</i> Collagen Membranes	<i>Zimmer</i> Socket Repair Membrane	<i>CopiOs</i> Pericardium Membrane <i>Puros</i> Pericardium Allograft Membrane	<i>Puros</i> Dermis Allograft Tissue Matrix
BENEFITS	Time-tested	Self-adhesive	Shape memory	Pre-cut for extraction sockets	Strong and conformable	User-friendly
RESORPTION*	10-14 days ³	24-48 hours** (Dissolves)	8 weeks (<i>BioMend</i>) ³ 18 weeks (<i>BioMend Extend</i>) ³	26-38 weeks ^{4***}	8-24 weeks ⁵	3-4 months (thin) 5-6 months (thick)

* Resorption rates may vary from patient to patient.

** May take up to 7 days.

*** When not exposed, the resorption rate is 26-38 weeks; if left exposed, resorption time is shorter.

1. Park SH, Wang HL. Mucogingival pouch flap for sandwich bone augmentation: technique and rationale. *Implant Dent.* 2005;14:349-356.
2. Wang HL, Shotwell JL, Itose T, Neiva RF. Multidisciplinary treatment approach for enhancement of implant esthetics. *Implant Dent.* 2005;14:21-29.
3. Refer to package insert.
4. Data on file with Collagen Matrix, Inc.
5. Rothamel D, Schwarz F, Sager M, Herten M, Sculean A, Becker J. Biodegradation of differently cross-linked collagen membranes: an experimental study in the rat. *Clin Oral Implants Res.* 2005;16(3):369-378.

©2011 Zimmer Dental Inc. All rights reserved. 1088, Rev. 2/11. Puros Allografts and CopiOs Pericardium Membrane are manufactured by RTI Biologics, Inc. and marketed by Zimmer Dental Inc. CollaPlug, CollaCote, CollaTape, BioMend and BioMend Extend Absorbable Collagen Wound Dressings are manufactured by Zimmer Dental Inc. by Integra LifeSciences Corporation. CollaPlug, CollaCote and CollaTape Absorbable Collagen Wound Dressings are registered trademarks of Integra LifeSciences Corporation. Zimmer Socket Repair Membrane is manufactured by Collagen Matrix, Inc. for Zimmer Dental Inc. Zimmer Collagen Capsule and Wedge and Zimmer CurV Pre-Shaped Collagen Membranes are manufactured by Osseous Technologies of America for Zimmer Dental Inc. HemCon is a registered trademark of HemCon Medical Technologies, Inc. HemCon Dental Dressing is manufactured by HemCon Medical Technologies, Inc. and distributed by Zimmer Dental Inc. Please note that not all products and regenerative materials are registered or available in every country/region and Puros and CopiOs part numbers may vary. Please check with a Zimmer Dental representative for availability and additional information.